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9 Maintenance and Troubleshooting

9.1 Cleaning and disinfecting

When cleaning the unit, keep in mind the following:

- Press the Power On/Off button so that the unit is off.
- Unplug the power plug from the power outlet before cleaning or disinfecting the unit.
- After each patient use, clean the accessories using a soft, clean cloth dampened with water and a mild antibacterial detergent. Avoid the use of abrasive materials and cleaning solvents.
- Wait until the unit is completely dry before operating it again.

9.1.1 Cleaning of apparatus

To clean the unit, turn it off and unplug the power supply. Clean the unit with a damp cloth. Do not use abrasive cleaners. A small amount of mild household detergent may be used, if needed.

9.1.2 Cleaning display panel

Use a soft and dry cotton cloth or micro fiber tissue to clean the panel. To remove fingerprints or grease, use a non-abrasive glass cleaning agent. Apply a small amount of the cleaning agent to a soft cotton cloth and then carefully clean the panel.

CAUTION:

- Do not spray the cleaning agent directly on the glass panel.
- Do not use cleaning agents that contain strong alkalis, lye, acid, detergents with fluoride or detergents with ammonia.

9.1.3 Laser applicator

To prevent corrosion, clean and dry the contact surface immediately after use. We further recommend cleaning the applicator and cable daily, using lukewarm water. Check the applicator and cable regularly for damage.

Danger:

- Under no circumstances may liquid penetrate the openings on the unit (e.g. the connecting sockets of the laser applicator). Therefore, do not use cleaning or disinfectant sprays.
- The unit and cables may not be sterilized using steam or gas.
- Never clean the unit with abrasives, disinfectants, or solvents that could

scratch the housing or LCD or otherwise damage the unit.

- In order to prevent electrical shock, unplug the power plug from the power outlet before cleaning or disinfecting the unit.

9.2 Safety Inspections

The following safety inspections must be performed on this unit. This must be done by persons who, based on training, knowledge or practical experience, are capable of conducting the inspections correctly and independently.

WARNING:

The service personnel must wear the protective glasses while inspecting the Laser therapy device.

9.2.1 Visual Inspection (Daily)

When performing daily inspections of the unit, pay particular attention the following areas of potential damage:

- Deformation of unit housing
- Power cable damage
- Laser Applicator connection sockets damage
- Laser Applicator cable damage

9.2.2 Function Test (Daily)

When performing daily inspections of the unit, pay particular attention the following areas of potential damage:

- Correct function of indicators
- Display of operating modes
- Patient Interrupt Button

NOTE: It is the responsibility of the health care facility to verify that the unit complies with the facility, local, and national Earth Leakage limits.

9.3 What to do when problems occur

We have found that most problems occur because of inadvertent operating errors. Therefore, when the LS2100 displays an error code, please check whether the operating instructions have been correctly followed.

If power indicator light doesn't work when the unit is turned on, check whether the unit is correctly connected to an active wall outlet and the power cord is connected to the unit.

9.4 Warning Messages, Error Messages

9.4.1 Software Error code

When the apparatus is turned on, it will first execute a self test. When an error is detected, both during the self test and during normal operation, a pop-up screen will appear on the display. When the error is displayed, all outputs will be disabled. When this situation occurs, remove all cables and switch the apparatus off and switch it on again. When the error re-appears, stop using the device and contact your supplier.

Error Code	Error Tips	What to do
218	SDcard read failure	Check whether the SDcard is connected correctly
219	Favorites is full, add failure	Nothing, The operation is invalid
224	Communicate invalid	Check whether the 20P Ribbon cable is connected correctly and firmly
227	The board temperature is too high	Check whether the cooling fan work normally.
228	Font file fails to load, use the default settings	Check whether the SDcard is connected correctly
707	The password is not correct	Check the password entered
708	Enter the new password twice inconsistent!	Enter the same password twice
709	The new password saved failure	Nothing, The operation is invalid
710	The password is changed successfully	Nothing, The operation is normal
711	No applicator is detected	Check whether the applicator is connected correctly
714	The interlock is not connected	Check whether the interlock is connected correctly
718	The temperature sensor of CHA applicator is abnormal	Stop using, and send to repair
719	The temperature sensor of CHB applicator is abnormal	Stop using, and send to repair

724	The temperature of CHA applicator is high, please wait!	Stop using the applicator, waiting for handle cooling
725	The temperature of CHA applicator is high, please wait!	Stop using the applicator, waiting for handle cooling
726	The temperature of CHA applicator is too high, output is stopped!	Stop using the applicator, waiting for handle cooling
727	The temperature of CHB applicator is too high, output is stopped!	Stop using the applicator, waiting for handle cooling
728	The applicator is not match	Use the corresponding handle

9.4.2 Troubleshooting

Problem	Probable Causes	Possible Remedies
LCD display fails to light	AC power cord contact failure	Ensure AC power cord is connected. Check the following contacts: <ul style="list-style-type: none"> • All contacts are in place. • All contacts are not broken. • Ensure that AC power cord is Connected.
The device cannot enter the ready state	The interlock does not connect correctly	Check whether the interlock is connected. Unplug the interlock, and then re-plugging it.
	The applicator does not connect correctly	Unplug the applicator, and then re-plugging it.
	The applicator is over heating	Stop use the applicator, and let it cool.
	The temperature sensor applicator is abnormal	Stop using, and send to repair.

9.4.3 Laser Applicator Error

The laser applicator has reported an error. Disconnect the device, wait some time and reconnect it. If the error persists, stop using the device and contact your supplier.

9.5 Technical Maintenance

On request a service manual can be made available containing: spare part list, descriptions, calibration instructions and other information which will assist the user's qualified technical personnel to repair those parts of the equipment which are designated by the manufacturer as repairable.

CAUTION:

- Electrical safety of the device relies on a properly earthed electrical connection via the power cord. It is therefore necessary to have this connection checked annually.
- To ensure continued compliance with the 21 CFR 1050.10 standard, this unit should be adjusted and safety tested once each year. Procedures laid down in the service manual should be followed. This may be carried out by your supplier, or by another agency, authorized by the manufacturer. It is also recommended that a service history record is maintained. In some countries this is even obligatory.
- Use of controls or adjustments or performance of procedures other than those specified herein may result in hazardous exposure to energy.

WARNING:

- This unit operates with high voltages. No attempt should be made to disassemble the unit.
- Maintenance and repair should be carried out by authorized personnel only.
- The manufacturer will not be held responsible for the results of maintenance or repairs by unauthorized persons.
- Take the necessary precautions, such as wear the protective glasses and ect, during maintenance to avoid possible exposure to hazardous laser radiation. All other technical maintenance is restricted to authorized NU-TEK maintenance personnel.

9.6 DISPOSAL

The LS2100 contains materials that can be recycled and/or are noxious to the environment. Specialized companies can dismantle the unit and sort out these materials. When you dispose of the unit, find out about local regulations concerning waste management.

9.7 Calibration

It is recommended to calibrate the laser source every 1 year. The device should be sent to the factory or a Trained Technician for this procedure.

Measuring equipment specification:

Optical power meter

- Wavelength Range: include 600nm~900nm
- Measuring range: include 10mW~500mW
- Resolution: <5%

The brief calibration procedure is as follows.

1. Choose a enclosed, secure room, and put the calibrate device and optical power meter on a steady position. Put on the laser protective glasses.
2. Set the optical power meter correctly (Such as wavelength, measuring range etc.).
3. Insert the applicator which is needed to be calibrated, and place it in calibration tooling. Turn on the laser device. Using administrator password to enter System Settings – Maintenance – Power Calibration, and select the corresponding calibration power button. Then pressing the button of applicator, and adjust the output power by using the central controller, until the display value of optical power meter is consistent with the set value.
4. Click the "Save" button after power adjustment. The interface will shows that the calibration data has been saved successfully and the power calibration process has been completed correctly.

10 Specifications

10.1 Technical features

Device	Power supply	100-240V~, 50/60Hz
	Maximum absorbed power	60VA
	Programmable treatment time	Up to 4 minutes
	Mode of operation	Continuous or pulsed
	Emission frequency	1Hz-10000Hz
	Pulsed mode	90%
	Output channels	2
	LCD back-lit display	Graphic 480 X 800 pixels TOUCH SCREEN
	Protocols in memory	10
	Number of protocols that can be saved in the user memory	100
	Laser classification	3B
	Remote interlock connector	1
	Key control	Passwords
	Command of execution of the treatment	Button
	Class of isolation / parts applied according to the rule EN 60601-1	Class I, Type B
	Dimensions (Width*Height *Depth)	197*240*285 mm
sound	Speaker	
Laser Applicator (AC8001)	Diode Specifications	650±30nm
	Maximum Power	200mW±20%
	Beam divergence	19° x 34°
	Nominal Optical Hazard Distance (NOHD)	40cm

Laser Applicator (AC8002)	Diode Specifications	850±30nm
	Maximum Power	200mW±20%
	Beam divergence	12° x 30°
	Nominal Optical Hazard Distance (NOHD)	30cm
Laser Applicator (AC8101)	Applicator	9 Diode Cluster Laser 540mW
	Diode Specifications	Four 650±30 nm (10 mW) LED Five 650±30 nm (100mW)Laser
	Maximum Power	540mW±20%
	Beam divergence	19° x 34°
	Nominal Optical Hazard Distance (NOHD)	60cm
Laser Applicator (AC8102)	Applicator	9 Diode Cluster Laser 1040mW
	Diode Specifications	Four 650±30 nm (10 mW) LED Five 850±30 nm (200mW)Laser
	Maximum Power	1040mW±20%
	Beam divergence	12° x 30°
	Nominal Optical Hazard Distance (NOHD)	60cm
Conditions of use	Temperature environment: 5 °C ~ 40 °C Relative humidity: 10% ~ 85% Atmospheric pressure: 700 ~ 1060hPa	
Storage / transport conditions	Temperature environment: -20 °C ~ 55 °C Relative humidity: 10% ~ 93% Atmospheric pressure: 700 ~ 1060hPa	
Maximum Permissible Exposure (MPE)	cornea (650nm)	10W•m ⁻²
	cornea (850nm)	20W•m ⁻²
	skin (650nm)	2000W•m ⁻²
	skin (850nm)	3991W•m ⁻²

10.2 Warranty

This Shenzhen Dongdixin Technology Co., Ltd. (hereinafter called the Company) product is warranted against defects in materials and workmanship for a period of two years from the date of shipment. The Company will at its option, repair or replace components which prove to be defective during the warranty period, provided that the repairs or replacements are carried out by the Company or its approved agents. All repairs to the Product must be performed by a service center certified by the Company. All repairs to the Product must be performed by a service center certified by the Company. Any modifications or repairs performed by unauthorized centers or groups will void this warranty.

The warranty period for applicators is one year (12 months).

For service under this warranty, the product must be returned by the buyer within the applicable warranty period to Company. Shipping charges to Company under this warranty must be paid by the buyer. The buyer must also include a copy of the sales receipt or other proof of the date of purchase. If the product is returned without proof of the date of purchase, it will be serviced as out-of-warranty product at Company's prevailing service rates.

This Warranty Does Not Cover:




- Alteration, misuse, or neglect of the product voids this warranty.
- Any malfunction or failure in the Product caused by product misuse, including, but not limited to, the failure to provide reasonable and necessary maintenance or any use that is inconsistent with the Product User's Manual.
- All damage which is due to repairs or tampering by the customer or unauthorized third parties or persons.
- Damage which has arisen during the transport from the company to the buyer or during transport to the service centre.
- Accessories which are subject to normal wear and tear.

Some states do not allow limitations on how long an implied warranty last, or the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to the buyer. This warranty gives the buyer specific legal rights, and buyer may also have other rights which vary from state to state.


11 Glossary of symbols

11.1 Glossary of General symbols

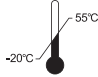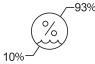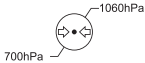	<p>Electrical devices are recyclable material and should not be disposed of with household waste after their useful life! Help us to protect the environment and save resources and take this device to the appropriate collection points. Please contact the organization which is responsible for waste disposal in your area if you have questions.</p>
	<p>Type B Applied Part</p>
	<p>Refer to instruction manual</p>
	<p>Please refer to instruction manual because of the higher levels of output</p>
	<p>EMERGENCY LASER STOP. To stop the laser output immediately in case of emergency.</p>
	<p>Date of manufacture</p>
	<p>The name and the address of the manufacturer</p>
	<p>Remote interlock connector</p>
	<p>Complies with the European Medical Device Directive(93/42/ECC)and amended by directive 2007/47/EC requirements. Notified body TUV Rheinland(CE0197)</p>
	<p>The name and the address of the Authorized EC-representative in Europe</p>

	<p>Transportation and storage temperature from -20 °C to 55 °C</p>
	<p>Transportation and storage humidity limits from 10% to 93%</p>
	<p>Transportation and storage atmospheric pressure limits from 700hPa to 1060hPa</p>

11.2 Glossary of Warning symbols

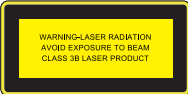	<p>Warning: The values of laser light energy used during therapy exceed safe values and proper precautions listed in this manual need to be taken.</p>
	<p>Warning: the values of supply voltage exceed safe values need to be taken.</p>
	<p>Warning: Avoid exposure to beam and indicated the classification of the device.</p>

12 Important information regarding electromagnetic compatibility (EMC)

With the increased number of electronic devices such as PC's and mobile(cellular) telephones, medical devices in use may be susceptible to electromagnetic interference from other devices. Electromagnetic interference may result in incorrect operation of the medical device and create a potentially unsafe situation.

Medical devices should also not interfere with other devices.

In order to regulate the requirements for EMC (Electro Magnetic Compatibility) with the aim to prevent unsafe product situations, the IEC60601-1-2 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices.

Medical devices manufactured by MANUFACTURER conform to this IEC60601-1-2 standard for both immunity and emissions.

Nevertheless, special precautions need to be observed:

- The use of accessories and cables other than those specified by MANUFACTURER, with the exception of cables sold by MANUFACTURER as replacement parts for internal components, may result in increased emission or decreased immunity of the device.
- The medical devices should not be used adjacent to or stacked with other equipment. In case adjacent or stacked use is necessary, the medical device should be observed to verify normal operation in the configuration in which it will be used.
- Refer to EMC table guidance regarding the EMC environment in which the device should be used.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Table 1: GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC EMISSIONS

Guidance and manufacturer's declaration - electromagnetic emissions		
The LasoRehab ² LS2100 device is intended for use in the electromagnetic environment specified below. The customer or the user of the LasoRehab ² LS2100 should assure that it is used in such an environment.		
Emissions tests	Conformity	Electromagnetic environment - guidance
RF emissions according to CISPR 11	Group 1	The LasoRehab ² LS2100 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions according to CISPR 11	Class A	The LasoRehab ² LS2100 device is equipment suitable for use in all establishments other than domestic and those directly connected to a low voltage power supply network which supplies buildings used for domestic purposes.
Harmonic emissions according to IEC 61000-3-2	Class A	
Voltage fluctuation emissions/flicker emissions according to IEC 61000-3-3	Complies	

Table 2: GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

Guidance and manufacturer's declaration — electromagnetic immunity			
The LasoRehab ² LS2100 device is intended for use in the electromagnetic environment specified below. The customer or the user of the LasoRehab ² LS2100 device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5kV, ± 1 kV line(s) to lines ± 0.5kV, ± 1 kV, ± 2 kV line(s) to earth	± 0.5kV, ± 1 kV line(s) to lines ± 0.5kV, ± 1 kV, ± 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT for 0.5 cycle 40% UT(60% dip in UT for 5 cycles) 70% UT (30% dip in UT for 25 cycles) <5% UT(>95% dip in UT for 5 seconds	<5% UT (>95% dip in UT for 0.5 cycle 40% UT(60% dip in UT for 5 cycles) 70% UT (30% dip in UT for 25 cycles) <5% UT(>95% dip in UT for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the use of the LasoRehab ² LS2100 device requires continued operation during mains power interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: UT is the a.c. mains voltage prior to application of the test level.			

Table 3: GUIDANCE AND MANUFACTURER’S DECLARATION ELECTROMAGNETIC IMMUNITY

Guidance and- manufacturer’s declaration. Electromagnetic immunity			
The LasoRehab ² LS2100 device is intended for use in the electromagnetic environment specified below. The customer or the user of the LasoRehab ² LS2100 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF disturbance variables according to IEC 61000-4-6	3 Vrms 0.15 MHz to 80 MHz	3 Vrms 0.15 MHz to 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of the LasoRehab ² LS2100 device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2\sqrt{P}$, 150 KHz to 80 MHz $d = 0.35\sqrt{P}$, 80MHz to 800MHz $d = 0.7\sqrt{P}$, 800MHz to 2.7GHz Where P is the maximum output power rating of the transmitter In watts (W) according to the. Transmitter manufacturer and d is the recommended separation distance in meters (m) ^b . Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur In the vicinity of equipment marked  with the following symbol:
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz	10 V/m 80 MHz to 2.7 GHz	
NOTE 1 At 80 MHz ends 800 MHz. the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a. Field strengths from fixed transmitters, such as base stations for radio (cellular/ cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an			

lectromagnetic site survey should be considered. If the measured field strength in the location in which the LasoRehab² LS2100 device is used exceeds the applicable RF compliance level above, should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the LasoRehab² LS2100.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Table 4: RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND ST-THERM LS2100

Recommended separation distances between portable and mobile RF communications equipment and the LasoRehab ² LS2100 device			
The LasoRehab ² LS2100 device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the LasoRehab ² LS2100 device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the LasoRehab ² LS2100 as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output Power of transmitter (W)	Separation distance according to frequency of transmitter(m)		
	150 kHz to 80 MHz $d= 1.2\sqrt{P}$	80 MHz to 800MHz $d= 0.35\sqrt{P}$	800 MHz to 2.7GHz $d= 0.7\sqrt{P}$
0.01	0.12	0.04	0.07
0.1	0.38	0.11	0.22
1	1.2	0.35	0.7
10	3.8	1.1	2.2
100	12	3.5	7.0
<p>For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p> <p>NOTE 1 At 80 MHz and 800 MHz. the separation distance for the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			

Shenzhen Dongdixin Technology Co., Ltd.
Floor 1-2, No.3 Building, Fanshen Xusheng Industrial Estate
Xilixiaobaimang 518108 Nanshan District, Shenzhen P. R. China
Tel: 0086-755-27652316
E-mail: service@nutekmedical.com

Authorized EC-representative:
Shanghai International Holding Corp.GmbH(Europe)
Eiffestraße 80,20537 Hamburg Germany
Tel:0049-40-2513175 Fax:0049-40-255726

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Всички права запазени. Ред. V2.1 © 202105